



## ST JOHNS FIRST AID KITS PRIVATE LIMITED

SL Plaza, Ayyencherry Main Road, Urappakkam - 603210

### Triangular Bandage



Product Name	:	SJF TB2
Product Code	:	Triangular Bandage
Specification	:	96x96x136, Safety Pin
MRP	:	Rs.60
Shelf Life	:	3 Years from Mfg Dt
Manufactured in India By	:	St Johns First Aid Kits Pvt Ltd
Email Id	:	<a href="mailto:info@stjohnsfirstaid.net">info@stjohnsfirstaid.net</a>
Whatsapp	:	8849004488
Web Site	:	<a href="https://www.stjohnsfirstaid.net">https://www.stjohnsfirstaid.net</a>
How to find Manufacturing Date	:	Refer the Lot No on the Pack. First 2 Digits are S.No. Second 2 Digits are Month of Manufacturing. Third 2 Digits are Manufacturing Year.

#### Application :

Triangular bandages can be used as large dressings, as slings to support a limb, or to secure a dressing in place.

If you're using a triangular bandage as a sling on an arm, you use it opened out. You should: Ask the person to hold their arm across their chest and support the arm while you work

put the bandage under the arm and around the back of the neck

put the other half of the bandage over the arm to meet at the shoulder and tie into a knot

tuck the loose ends of the bandage in at the elbow, or use a pin

If you're using a triangular bandage to support a lower limb or large dressing, fold it in half horizontally so the point of the triangle touches the middle of the long edge.

Then fold it in half again in the same direction to make a broad strip